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the eyelid to return the lens to the central position. If this problem persists, consult your eye care professional. Never use tweezers, suction cups, sharp objects or your finger nails to remove your lenses from the lens container or your eyes.

FALLING ASLEEP WHILE WEARING LENSES If you inadvertently fall asleep for a short period while wearing your lenses no adverse effect are likely. The lenses may however feel dry, this can usually be relieved by blinking fully several times.

TAKING CARE OF YOUR LENSES

- CORRECT CARE OF YOUR LENSES IS ESSENTIAL.
- DISINFECT your lenses each time they are removed to destroy harmful germs and to ensure safe and comfortable contact lens wear
- A separate **CLEANING** and **RINSING** step may be recommended by your eye care professional to remove contaminants from the lens surface. In addition, your lenses may require periodic enzymatic cleaning to remove protein.
- Your eye care professional should choose the lens care system most suitable for you.
- Ask your eye care professional before using any alternative lens care products.
- Do not alternate or mix lens care products from different systems.
- Use only soft contact lens care products following the manufacturer's instructions.
- DO NOT USE HEAT NOR ANY PRODUCT DESIGNED FOR HARD OR RIGID GAS PERMEABLE LENSES.

Always remember the following:

- Always wash, rinse and dry your hands before handling your contact lenses.
- Never store lenses overnight in saline only. Saline will NOT protect your lenses from germs and your eyes from possible infection.
- Never re-use the solution in your lens case.
- Never use solutions that are past their expiry date.
- Rinse your lens case every time you remove your lenses from it. Use sterile saline or fresh lens care solution and, if appropriate, allow to air dry. This helps avoid contamination and serious eye injury.
- Replace your lens case regularly.

IF YOU HAVE NOT WORN YOUR LENSES FOR A FEW DAYS, follow the instructions given for your contact lens care regimen and, if applicable, clean and disinfect your lenses prior to insertion.

WHEN NOT TO WEAR LENSES Contact lenses should not be worn in conjunction with certain health or environmental conditions. Conditions that may prevent or interfere with safe contact lens wear include:

- Allergy, inflammation, infection or irritation in or around the eye or eyelids.
- Periods of poor health, such as severe colds or influenza.
- The use of some medications, including eye medication.
- Inadequate tear film (dry eyes).
- Excessively dry or dusty environments that make contact lens wear uncomfortable.
- Water sport without the use of goggles.

Consult your eye care professional specifically regarding these or other conditions.

POSSIBLE PROBLEMS Although soft contact lenses provide many benefits to the wearer, it is possible that problems might occur and may be first noticed as one or more of the following signs:

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PHEMFILCON A and VIFILCON A DAILY WEAR MONTHLY REPLACEMENT SOFT CONTACT LENSES

PRODUCT NAMES

FRESHLOOK® COLORBLEND® soft contact lenses, **FRESHLOOK® COLORS** soft contact lenses and **FRESHLOOK® DIMENSIONS** soft contact lenses contain 45% phemfilcon A (with UV absorbing chromophores), and 55% water.

FRESHLOOK® COLORBLEND® contact lenses and **FRESHLOOK® COLORS** contact lenses and **FRESHLOOK® DIMENSIONS** contact lenses are packaged in individual blisters containing isotonic, borate buffered saline. The saline contains 0.005% Poloxamer. **FRESHLOOK® COLORBLEND®** contact lenses and **FRESHLOOK® COLORS** contact lenses are printed with an intermittent coating containing a combination of the following pigments: iron oxides, titanium dioxide, [phthalocyaninato (2-)] copper, chromium oxide, carbazole violet, phthalocyanine green. **FRESHLOOK® DIMENSIONS** contact lenses are tinted with phthalocyanine green and are printed with an intermittent coating containing a combination of the following pigments: iron oxides, [phthalocyaninato (2-)] copper, chromium oxide, carbazole violet, phthalocyanine green.

FOCUS® Monthly VISITINT® soft contact lenses contain 45% vilcon A and 55% water. The lenses are tinted light blue to assist handling; the tint contains Remazol Brilliant Blue R.

FOCUS® Monthly VISITINT® contact lenses are packaged in individual blisters containing isotonic, phosphate buffered saline. The lenses are steam sterilised **STERILE**[ⓘ]. Use only if printed foil pack is intact.

INTENDED USE

All phemilcon A and vilcon A contact lenses are intended for daily wear in one or both eyes for the optical correction of vision in persons with non-diseased eyes with minimal astigmatism that does not interfere with vision.

RECOMMENDED WEAR AND REPLACEMENT SCHEDULE DAILY WEAR - less than 24 hours while awake. The wearing time may be built up gradually until the maximum recommended is achieved. THE LENSES MUST BE REMOVED BEFORE SLEEPING. In the interests of maintaining the health of your eyes, the wearing schedule should be determined by your eye care professional. We recommend that your lenses be discarded and replaced with a new pair each month.

HANDLING YOUR LENSES

Your eye care professional should provide you with detailed contact lens insertion and removal instructions. Always wash and dry your hands before handling lenses. *Gently shake the blister pack before opening.* Peel back the foil lid. Pour the lens into the palm of your hand or if necessary, gently slide the lens out of the container using your forefinger. Ensure the lens is not inside out and that you have the appropriate lens for that eye. Inspect your lenses prior to insertion. Do not insert if damaged. When removing your lens, make sure that your hands are completely dry. Blink fully several times, then while looking up, slide the lens down onto the white part of the eye. Remove the lens by gently pinching it between the thumb and forenger. If the lens decentres on the eye, close your eye and gently massage

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© 2014 Novartis