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<sup>1</sup>Cutter GR, Chalmers RL, Roseman M. The Clinical Presentation, Prevalence, and Risk Factors of Focal Corneal Infiltrates in Soft Contact Lens Wearers .*The CLAO Journal*. Jan 1996; 22 (1): 30-37.  
<sup>2</sup>Schein OD, Glynn RJ, Poggio EC, Seddon JM, Kenyon KR. The Relative Risk of Ulcerative Keratitis Among Users of Daily-Wear and Extended-Wear Soft Contact Lenses. *N Eng J Med*. 773-83; (12)321; 1989

INSTRUCTIONS FOR USE

THIS LEAFLET CONTAINS IMPORTANT PRODUCT USE AND SAFETY INFORMATION. PLEASE READ CAREFULLY AND RETAIN FOR FUTURE REFERENCE. CONTACT LENSES SHOULD ALWAYS BE FITTED BY AN EYE CARE PROFESSIONAL. IT IS ESSENTIAL THAT YOU FOLLOW YOUR EYE CARE PROFESSIONAL'S DIRECTIONS AND ALL LABELING INSTRUCTIONS FOR PROPER USE OF YOUR CONTACT LENSES.

PRODUCT IDENTIFICATION (Name)

- Toric Lenses
  - DAILIES TOTAL1™ for Astigmatism (delefilcon A) Water Gradient One-Day Soft Contact Lenses

PRODUCT DESCRIPTION

The lens material is approximately 33% water and 67% delefilcon A, a silicone containing hydrogel with added phosphatidylcholine. The color additive copper phthalocyanine is added to the lens material to create a light blue-green edge-to-edge color (handling tint) to make it easier to see when handling. Benzotriazole UV and UV-Vis absorbing monomers are used to block UV radiation and reduce transmittance of high energy visible light (HEVL) wavelengths in the range from 380 nm to 450 nm. The transmittance characteristics are less than 1% in the UVB range of 280 nm to 315 nm and less than 10% in the UVA range of 315 nm to 380 nm for the entire power range. The thinnest lenses block 33% of radiation across the blue-light wavelength in the range from 380 nm to 450 nm. The core lens material containing 33% water transitions through a water gradient to a hydrogel surface layer that exceeds 80% water. To aid the fitting process, DAILIES TOTAL1™ for Astigmatism (delefilcon A) toric contact lenses feature a scribe line on the front lens surface to enable assessment of the lens orientation.

TECHNICAL PERFORMANCE CHARACTERISTICS

Lens Properties

|                              |                                                                                                                                           |
|------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------|
| Refractive Index (hydrated): | 1.42                                                                                                                                      |
| Light Transmittance:         | 90% ± 5% for average over 380 to 780 nm                                                                                                   |
| HEVL Transmittance:          | ≤ 80%T at 420 nm                                                                                                                          |
| UV Transmittance:            | τUVB < 1.0 % (average percent transmittance over 280 nm to 315 nm)<br>τUVA < 10.0 % (average percent transmittance over 315 nm to 380 nm) |
| Oxygen Permeability (Dk):    | 140 barrer units*, measured at 35 °C (intrinsic Dk - Coulometric method)                                                                  |
| Water Content:               | 33% by weight in normal saline                                                                                                            |
| Surface Water Content:       | ≥ 80%                                                                                                                                     |

Available Lens Parameters

Toric Lenses:

- Chord Diameter: 14.5 mm
- Center Thickness: 0.11 mm @ -3.00 D (varies with power)
- Base Curves and Powers:
  - Base Curve: 8.6 mm
    - Plano
    - Minus Powers: -0.25 D to -6.00 D (0.25 D steps)  
-6.50 D to -8.00 D (0.50 D steps)
    - Plus Powers: +0.25 D to +4.00 D (0.25 D steps)
    - Cylinder Powers: -0.75 D, -1.25 D, -1.75 D, -2.25 D
    - Axis: 10° to 180° (10° steps)

ACTIONS (Mode of Action)

When hydrated and placed on the cornea, delefilcon A one-day soft contact lenses act as a refracting medium to focus light rays on the retina.

Reduced dryness symptoms:

Delefilcon A water gradient one-day soft contact lenses provide reduced dryness symptoms and longer comfortable wearing time in contact lens wearers who experience discomfort symptoms related to dryness during lens wear.

UV and UV-Vis filtering:

The lenses contain a combination of UV and UV-Vis blocking monomers to help protect against transmission of harmful UV radiation to the cornea and into the eye. For example, a lens with 0.09 mm center thickness (~3.00 D, thinnest projected parameter) blocks 97% of UVA radiation, and 99% of UVB radiation averaged across the spectrum. The lenses reduce high energy visible light (HEVL) radiation reaching the back of the eye by about 33% in the range from 380 nm to 450 nm. The radiation transmittance will be further reduced with increasing lens thickness.

**WARNING: UV absorbing contact lenses are NOT substitutes for protective UV absorbing eyewear, such as UV absorbing goggles or sunglasses because they do not completely cover the eye and surrounding area. The patient should continue to use UV absorbing eyewear as directed.**

**NOTE:** Long-term exposure to UV radiation is one of the risk factors associated with cataracts. Exposure is based on a number of factors such as environmental conditions (altitude, geography, cloud cover) and personal factors (extent and nature of outdoor activities). UV-Blocking contact lenses help provide protection against harmful UV radiation. However, clinical studies have not been done to demonstrate that wearing UV-Blocking contact lenses reduces the risk of developing cataracts or other eye disorders. High energy visible light filtering provided by delefilcon A soft contact lenses with light absorbing chromophores is additive to the natural crystalline lens. There is no demonstrated clinical benefit to a 33% reduction in visible light at wavelengths below 450 nm. The eye care professional should be consulted for more information.

INTENDED PURPOSE / USE

Delefilcon A soft contact lenses (toric) are intended for on-eye use in persons with healthy eyes that need vision correction as determined by an eye care professional.

Delefilcon A soft contact lenses (toric) are intended for single use, daily disposable wear (less than 24 hours while awake) only. The lenses are not intended to be cleaned or disinfected and should be discarded after a single use.

| Intended Use Specific to Lens Design (Model) |                                                                                                |
|----------------------------------------------|------------------------------------------------------------------------------------------------|
| Lens Design (Model)                          | Intended Use (On-eye)                                                                          |
| Toric                                        | Optical correction of refractive ametropia (myopia and hyperopia) with additional astigmatism. |

PATIENT TARGET GROUPS

The intended wearer population for delefilcon A one-day soft contact lenses (toric) includes those persons with non-diseased eyes who require vision correction for refractive ametropia (myopia and hyperopia, with additional astigmatism), can be properly fitted with the device and are capable of understanding and following the wear, care, and safety instructions for use (or have a caregiver who can act on their behalf). The contact lenses are commonly worn by adults and teenagers. Children have worn contact lenses under the oversight of a qualified eye care professional and with parental supervision.

CLINICAL BENEFITS

Delefilcon A one-day soft contact lenses provide on-eye optical correction for myopia, hyperopia and astigmatism, as determined and fitted by an eye care professional.

Contact lenses with corrective power provide functional advantages over spectacles by improving peripheral (side) vision and by minimizing image size differences that result when the lens powers between the eyes are not the same.

INDICATIONS (USES)

DAILIES TOTAL1™ for Astigmatism (delefilcon A) soft contact lenses are indicated for the optical correction of refractive ametropia (myopia and hyperopia) in phakic or aphakic persons with non-diseased eyes with up to 6.00 diopters (D) of astigmatism.

The lenses are to be used for single use, daily disposable wear (less than 24 hours while awake) only.

CONTRAINDICATIONS (Reasons Not to Use)

Contact lenses should not be worn in conjunction with certain health or environmental conditions. Conditions that may prevent or interfere with safe contact lens wear include:

- Allergy, inflammation, infection or irritation in or around the eye or eyelids
- Inadequate tear film (dry eye)
- Corneal hypoesthesia (reduced corneal sensitivity)
- Use of any medication that is contraindicated or interferes with contact lens wear, including eye medications
- Any systemic disease which may be exacerbated by or interferes with contact lens wear
- If eyes become red or irritated

Consult your eye care professional specifically regarding these or other conditions.

WARNINGS

- Daily wear lenses are not indicated for overnight wear. Delefilcon A lenses should not be worn while sleeping.
- Serious eye problems, including corneal ulcers (ulcerative keratitis), can develop rapidly and lead to loss of vision.
- Contact lens wear increases the risk of eye infections. Sleeping in lenses and/or smoking further increases the risk of ulcerative keratitis for contact lens users.<sup>1,2</sup>
- If a patient experiences eye discomfort, foreign body sensation, excessive tearing, vision changes, or redness of the eye, the patient should be instructed to immediately remove lenses and promptly contact an eye care professional.
- Problems with contact lenses and lens care products could result in serious injury to the eye. It is essential to follow the eye care professional's directions and all labeling instructions for proper use of lenses and lens care products.

\*1 barrer unit: 1 x 10<sup>-11</sup> (cm<sup>2</sup>/sec) (ml O<sub>2</sub>/ml x mm Hg)

- UV absorbing contact lenses are NOT substitutes for protective UV absorbing eyewear, such as UV absorbing goggles or sunglasses because they do not completely cover the eye and surrounding area. The patient should continue to use UV absorbing eyewear as directed.

PRECAUTIONS

Special Precautions for the Eye Care Professional

- When selecting an appropriate lens design and parameters, the eye care professional should consider all characteristics of the lens that can affect lens performance and ocular health, including oxygen permeability, central and peripheral thickness and optic zone diameter.
- Trial lenses for fitting and diagnostic purposes should be discarded after a single use and not be re-used from patient to patient.
- The ocular health of the patient and lens performance on the eye should be carefully evaluated on initial dispensing and monitored on an ongoing basis by the prescribing eye care professional.
- Fluorescein, a yellow dye, should not be used while the lenses are on the patient's eyes. The lenses absorb this dye and become discolored.
- Patients who wear contact lenses to correct presbyopia may not achieve the best corrected visual acuity for either far or near vision. Vision requirements vary with the individual and should be considered when selecting the most appropriate type of lens for each patient.
- Diabetics may have reduced corneal sensitivity and thus are more prone to corneal injury and do not heal as quickly or completely as non-diabetics.
- Visual changes or changes in lens tolerance may occur during pregnancy or use of oral contraceptives. Caution patients accordingly.
- Eye care professionals should instruct the patient to remove the lenses immediately if the eye becomes red or irritated.
- Before leaving the eye care professional's office, patients should be able to promptly remove their lenses or should have someone else available who can remove their lenses for them.
- Routine eye examinations are necessary to help assure the continued health of the patient's eyes. Alcon recommends that patients see their eye care professional once each year, or more often, as recommended by the eye care professional.

Lens Handling and Wearing Precautions

- Check eyes daily to ensure they look well, feel comfortable and your vision is clear.
- Do not use if blister pack is damaged or not sealed completely. This may result in product contamination which can lead to a serious eye infection.
- If a blister foil pack is unintentionally opened before the lens is intended for use, the lens should be discarded. Product contamination may result, which can lead to a serious eye infection.
- In the interest of maintaining the health and safety of your eyes, the lens wearing schedule should be determined by an eye care professional.
- The lenses are not intended to be cleaned or disinfected and should be discarded after a single use. Reuse can increase the risk of infection or fatigue-induced device failure.
- Do not share lenses with anyone as this may spread microorganisms which could result in serious eye health problems.
- Never allow contact lenses to come into contact with non-sterile liquids (including tap water and saliva) as microbial contamination can occur, which may lead to permanent eye damage.
- Consult your eye care professional before wearing lenses during sporting activities including swimming and water related activities. Exposure to water (or other non-sterile liquids) while wearing contact lenses in activities such as swimming, water skiing, and hot tubs may increase the risk of ocular infection, including but not limited to *Acanthamoeba* keratitis.
- Remove and discard lenses when exposed to noxious or irritating vapor.
- Discard a contact lens that has become dehydrated or damaged. Replace with a fresh new lens.
- It is recommended that contact lens wearers visit their eye care professional at least once per year, or as directed.
- Inform employer that you wear contact lenses, especially if job involves using eye protection equipment.
- Keep a note of the correct lens power for each eye. Before lens insertion, check that the lens power on each foil pack is correct for that eye.
- Do not change the lens type or parameters without first consulting an eye care professional.
- If lens wear is discontinued for a length of time, consult your eye care professional before restarting wear.
- Be careful while using soaps, lotions, creams, cosmetics or deodorants as they can cause irritation if they come into contact with your lenses.
- Insert lenses before applying make-up and remove them before taking make-up off.
- Always carry spare lenses with you or have back-up spectacles available.
- Do not use lenses after the expiry date.

ADVERSE REACTIONS (Possible problems and what to do)

Problems may occur with contact lens wear and may first present as one or more of the following signs and symptoms:

- Feeling of something in the eye (foreign body sensation)
  - Uncomfortable lens
  - Eye redness
  - Sensitivity to light (photophobia)
  - Burning, stinging, itching or watering eyes
  - Reduced sharpness of vision
  - Rainbows or halos around lights
  - Increased eye secretions
  - Discomfort or pain
  - Severe or persistent dry eyes
- These symptoms, if ignored, may lead to more serious complications.

WHAT TO DO IF A PROBLEM OCCURS

If any of the above signs or symptoms occur, immediately remove the lens(es).

- Inspect the affected eye(s) in a mirror, looking for signs of irregularity.
  - If the discomfort or problem stops, insert a fresh, new lens.
  - If the discomfort or problem does not stop or if it returns after insertion of a new lens(es), remove the lens(es) and immediately contact your eye care professional.

A serious condition such as infection, corneal ulcer (ulcerative keratitis) or iritis may be present. These conditions could progress rapidly and may lead to permanent loss of vision. Less serious reactions such as abrasions, epithelial staining and bacterial conjunctivitis must be managed and treated properly to avoid complications.

- Occasional dryness may be relieved by blinking fully several times or by the use of contact lens rewetting drops that are approved for use with soft contact lenses. If dryness persists, consult your eye care professional.
- If a lens sticks (stops moving), apply several drops of a contact lens rewetting solution and wait until the lens begins to move freely on the eye. If this problem persists, consult your eye care professional.
- If a lens decenters on the eye, it may be possible to recenter it by:
  - Closing your eyelids and gently massaging the lens into place, or
  - Looking in the direction of the lens and blinking gently, or
  - Gently pushing the off-centered lens onto the cornea with light finger pressure on the edge of the upper or lower eyelid.
- If a lens tears in your eye, remove the pieces carefully by pinching them as you would for normal lens removal. If the lens pieces do not seem to remove easily, do not pinch the eye tissue. Rinse with sterile saline and try to remove the lens pieces again. If this does not help, contact your eye care professional for assistance.

General Emergencies:

If chemicals of any kind (household products, gardening solutions, laboratory chemicals, etc.) are splashed into the eyes:

- Flush eyes immediately with saline solution or tap water.
- Remove and discard the lenses and immediately contact your eye care professional or visit a hospital emergency room without delay.

SERIOUS INCIDENT REPORTING

Any serious incident related to the use of this medical device should be reported to Alcon Laboratories, Inc.:

Contact the local country office or your Alcon distributor

Email: [qa.complaints@alcon.com](mailto:qa.complaints@alcon.com)

Website: <https://www.alcon.com/contact-us>

Serious incidents should also be reported to the competent authority for medical devices of your State.

DIRECTIONS FOR USE

Start each day with a fresh pair of lenses that are discarded at the end of every daily wearing period.

LENS HANDLING INSTRUCTIONS

- Always wash and rinse your hands thoroughly and dry with a clean, lint-free towel before handling contact lenses.
- Shake the blister pack gently prior to opening.
- Remove the lens from the blister pack by carefully pouring it onto the palm of your hand.
- Ensure the lens is right side out and that you have the correct lens for each eye.
- Inspect the lenses prior to insertion.
- Do not insert damaged or unclear lenses.

LENS INSERTION INSTRUCTIONS

- Wash and rinse your hands thoroughly and dry with a clean, lint-free towel before handling contact lenses.
- Place a lens on the tip of your clean and dry right or left index finger. Place the middle finger of the same hand close to lower eyelashes and pull down the lower eyelid.
- Use the fingers of the other hand to lift the upper eyelid.
- Place the lens directly on the eye (cornea) and gently roll finger away from the lens.
- Look down and slowly release the lower lid.
- Look straight ahead and slowly release the upper lid.
- Blink gently.

LENS REMOVAL INSTRUCTIONS

- Wash and rinse your hands thoroughly, and dry completely with a clean, lint-free towel.

- Blink fully several times.
- While looking up, use the tip of your finger to slide the lens down onto the white part of the eye.
- Remove the lens by pinching gently between the thumb and forefinger. Do not pinch the eye tissue.
- If the lens is difficult to grasp, dry your fingers once more and try again. Do not use rewetting drops in this instance.
- Never use tweezers, suction cups, sharp objects or your fingernails to remove lenses from the lens container or your eyes.

LENS CARE SOLUTIONS

Delefilcon A daily disposable contact lenses are not intended for use with a lens care system. The lenses are for single use, daily disposable wear only. They are not intended to be cleaned or disinfected and should be discarded after a single use. Keep replacement lenses or back-up spectacles available at all times.

If you experience problems with lens removal (lens decentered or damaged), see **WHAT TO DO IF A PROBLEM OCCURS** above.

DISPOSAL AND RECYCLING

Dispose of contact lenses and the blister pack lidding in the waste bin, not down the sink or toilet. The carton packaging and the polypropylene (PP) plastic shell of the blister pack should be placed in the waste bin or recycled according to local waste management guidance.

CONTENTS OF CONTAINER (How Supplied)

Each lens is packaged in a foil-sealed plastic blister pack containing phosphate buffered saline solution with approximately 0.3% of polymeric wetting agents consisting of copolymers of polyamidoamine and poly(acrylamide-acrylic acid), and is steam sterilized.

The package is marked with the base curve, diameter, dioptric power, cylinder power and axis (where applicable), manufacturing lot number, date of manufacture, and expiration date. Lenses are supplied sterile in foil-sealed blister packs. Finished lenses in individually sealed blister pack containers (strips of 5) are packed into cartons containing up to 90 individually sealed contact lenses.

ABBREVIATIONS AND SYMBOLS THAT MAY BE USED ON LABELING

| ABBREVIATION / SYMBOL                                                               | DEFINITION                                                                                                       |
|-------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------|
| BC                                                                                  | Base Curve                                                                                                       |
| DIA                                                                                 | Diameter                                                                                                         |
| PWR                                                                                 | Power                                                                                                            |
| D                                                                                   | Diopter (lens power)                                                                                             |
| L                                                                                   | Left                                                                                                             |
| R                                                                                   | Right                                                                                                            |
| UV                                                                                  | Ultra-violet                                                                                                     |
| UVA                                                                                 | Ultra-violet A                                                                                                   |
| UVB                                                                                 | Ultra-violet B                                                                                                   |
| UV-Vis                                                                              | Ultra-violet and Visible                                                                                         |
| HEVL                                                                                | High Energy Visible Light (blue light)                                                                           |
| CYL AXIS                                                                            | Cylinder power and axis                                                                                          |
|    | Packaging waste license sign                                                                                     |
|    | Do not re-use                                                                                                    |
|    | Batch code                                                                                                       |
|   | Use-by date                                                                                                      |
|  | Expiration date (Use-by date)                                                                                    |
|  | Single sterile barrier system                                                                                    |
|  | Sterilized using steam                                                                                           |
|  | European conformity mark                                                                                         |
|  | Two letter code for the language (Example shown: English)                                                        |
|  | Caution                                                                                                          |
|  | Consult instructions for use                                                                                     |
|  | Do not use if blister package is damaged                                                                         |
|  | Manufacturer                                                                                                     |
|  | Date of manufacture                                                                                              |
|  | Do not dispose lenses in toilet or sink                                                                          |
|  | Medical device                                                                                                   |
|  | Authorized Representative in the European Community                                                              |
|  | Caution: Federal law (USA) restricts this device to sale by or on the order of a licensed eye care professional. |

**Manufacturer:**  
Alcon Laboratories, Inc.  
6201 South Freeway  
Fort Worth, TX 76134-2099, USA

**Authorized Representative in the European Union:**  
Alcon Laboratories Belgium  
Lichterveld 3  
2870 Puurs-Sint-Amands, Belgium

**Importer and Registration Holder:**  
Lapidot Medical Import and Marketing Ltd.,  
8 Hashita St., Caesarea  
Telephone number: 1700-70-33-30  
Approved by the Ministry of Health, Medical Device Department. Registration No.: 4350685

Date of Issue:

© 2022 Alcon Inc.

REFERENCES

<sup>1</sup> Cutter GR, Chalmers RL, Roseman M. The Clinical Presentation, Prevalence, and Risk Factors of Focal Corneal Infiltrates in Soft Contact Lens Wearers. *The CLAO Journal*. Jan 1996; 22 (1): 30-37.

<sup>2</sup> Schein OD, Glynn RJ, Poggio EC, Seddon JM, Kenyon KR. The Relative Risk of Ulcerative Keratitis Among Users of Daily-Wear and Extended-Wear Soft Contact Lenses. *N Eng J Med*. 1989; 321(12):773-83.

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